43746

Avicenum PHLEBO 70 Calf-Length Stockings	
Size	Shoe size (EU)
S	36–38
M	39–41
L	42–44

Avicenum PHLEBO 70 Thigh-High Stockings		
Size	Shoe size (EU)	Thigh circumference (cm)
S	36–38	50–55
M	39–41	55–60
L	42–44	60–66
XL	45–47	66–72

Avicenum PHLEBO 70 Tights					
small gusset (G1)		large gusset (G2)		extra large gusset (G4)	
Size	full height/hip circumference	Size	full height/hip circumference	Size	full height/hip circumference
S	158–164/100–108	S+	158–164/116–124	-	-
M	164–170/100–108	M+	164–170/116–124	M	164–170/116–124
L	170–176/108–116	L+	170–176/116–124	L	170–176/116–124
XL	176–182/108–116	XL+	176–182/116–124	XL	176–182/116–124

Avicenum PHLEBO 70 TRAVEL Calf-Length Stockings	
Size	Shoe size (EU)
S	36–38
M	39–41
L	42–44
XL	45-47

(EN) INSTRUCTIONS FOR USE - Avicenum PHLEBO 70

Avicenum PHLEBO 70 are compression support stockings. They were made by circular knitting technology. The materials used in our compression stockings are kind to the skin and do not include latex. This compression medical device works by applying precisely defined, graduated pressure to the leg, with a gradual reduction in force from the ankle towards the heart. The pressure in the ankle area is highest, with values of up to 10–17 mmHg. For the desired effect, the right size must be used.

Name of the medical device:

Avicenum PHLEBO 70 calf-length stockings

Medical device variants:

- closed toe, size 36–38, 39–41, 42–44

Avicenum PHLEBO 70 thigh-high stockings

Medical device variants:

- closed toe, lace, size S–XL

Avicenum PHLEBO 70 tights

Medical device variants:

- closed toe, small gusset, sizes S, M, L, XL
- closed toe, large gusset, sizes S+, M+, L+, XL+
- closed toe, extra large gusset, sizes M, L, XL

Avicenum PHLEBO 70 TRAVEL calf-length stockings

Medical device variants:

- closed toe, size 36–38, 39–41, 42–44, 45–47

User profile:

Avicenum PHLEBO 70 compression stockings are a medical device intended for use both at home and in medical care facilities.

Intended use of the medical device:

This medical device is intended for compression therapy of circulatory disorders and various types of oedema in the leg.

Indications:

- Chronic venous insufficiency – its subjective and objective manifestations, prevention

- Primary and secondary varicose veins in the legs
 - Superficial thrombophlebitis
 - Oedemas associated with post-thrombotic syndrome after overcoming phlebothrombosis
 - Oedemas and varicose veins in pregnancy
 - Conditions following the occurrence of phlebothrombosis and thrombophlebitis in the legs
 - Prevention of phlebothrombosis in surgical treatment and subsequently also thromboembolic conditions (especially surgery, orthopaedics, gynaecology)
 - Oedemas of the lower limbs (postoperative and post-traumatic, lymphatic, obesity-related oedemas, idiopathic oedemas of the lower limbs)
 - Compression support following sclerotherapy of varicose veins of the legs
 - Compression support following surgical extirpation of varicose veins (classic surgery, radiofrequency ablation or laser ablation of varicose veins)
 - Travel-related thrombosis (economy class syndrome)
- If you are using compression stockings for the first time, consult your doctor to determine the proper compression class and have regular check-ups during treatment.

Contraindications:

- Acute weeping skin manifestations
- Acute limb ischaemia
- Critical limb ischaemia
- Decompensated cardiac insufficiency
- Phlegmasia cerulea dolens, phlegmasia alba dolens
- Hypersensitivity to materials used
- Limb numbness disorders (e.g. peripheral neuropathy in diabetes mellitus)

In view of known contraindications your doctor should evaluate possible risks and weigh them against the therapeutic benefits of use when considering application of compression stockings.

Patient target group:

Patients with primary varicose veins, patients who have experienced venous thrombosis and patients with the above-mentioned indications. Compression medical stockings are intended for use on healthy, intact skin. This compression medical device is designed for all age groups and sexes. There are no restrictions for use during pregnancy. In pregnancy, it is recommended to use tights with an extra large gusset (G4).

Side effects:

Side effects can occur, especially if the belt is not fitted correctly. There may be bruising, impaired blood flow due to pressure on blood vessels, or impaired sensitivity due to excessive nerve compression. Sensitive individuals may experience skin irritation (redness, itching, or blistering) in places where the compression stocking comes into contact with the skin.

How to choose the right size compression stockings:

For the desired effect, the right size must be used. The size can be determined from the size chart.

Warning:

Any oedema of the lower limbs should be examined by your doctor. If this is the first time you experience oedema, consult your doctor before using the product. Do not use the product if you have not determined the cause of the oedema. Always consult your doctor before using this product in combination with other medical devices (such as bandages or orthoses).

If the measured value is between sizes, choose the larger one. If you are unsure about the size of the compression stockings, visit a specialist medical supply store or contact the manufacturer directly.

If compression stockings are difficult to put on, visit a specialist medical supply store and ask for a demonstration of the correct technique for putting on compression stockings. You can also use one of the aids for putting on the product.

Correctly chosen size:

- You should feel noticeable but not unpleasant pressure over the full length of the compression stocking
- The stocking should not slide down when walking, nor bunch up under the knee or at the instep
- The welt or lace band on the stocking is not too tight

Putting on your compression medical stockings:

Ease the compression stockings onto the lower limb gradually, not by pulling on the upper edge alone. It is best to put on compression stockings in the morning before any swelling develops. When pulling on the compression stockings, keep them away from sharp objects (e.g. jewellery) and long nails. Increased care of the soles of your feet (rough skin on the heels) will reduce the risk of damaging the product. Compression stockings must be stretched evenly and without folds to distribute the pressure effectively. If you use body cosmetics, wait to put on compression stockings for about 15 minutes after application. In the event of visible mechanical damage to the product, stop wearing it and replace it with a new one.

Care and maintenance:

Avicenum PHLEBO 70 stockings are medical devices intended for repeated use by a single user. This product may be used safely and effectively for 6 months from the first use if the following guidelines are respected:

- Medical socks can be washed in a washing machine (maximum wash temperature 30°C, gentle cycle), preferably in a protective cover (wash bag)
- Do not use fabric softener
- Lay flat to dry away from sources of direct heat (radiator, sunlight)
- Do not iron medical devices
- Maximum number of washing cycles is 120
- Store in a dark, dry place in the original packaging if possible
- Do not make mechanical changes to the medical device, do not repair areas damaged by use, do not trim the edges
- Keep the medical device away from organic solvents and bleaching agents

This medical device may be used safely for 60 months from the date of manufacture. The expiry date is marked on the packaging of the medical device.

Care symbols:

Do not use fabric softeners

When not to use:

Stop using the product immediately if any of the following conditions apply:

- There are signs of mechanical damage – replace with new compression medical stockings
- You notice a deterioration in your health that may be associated with the use of the compression medical device – consult your doctor in such case

Composition:

Avicenum PHLEBO 70 Calf-Length Stockings: 76% PA, 24% elastane
 Avicenum PHLEBO 70 Thigh-High Stockings: 76% PA, 24% elastane, silicone
 Avicenum PHLEBO 70 Tights: 76% PA, 24% elastane
 Avicenum PHLEBO 70 TRAVEL Calf-Length Stockings: 76% PA, 24% elastane

This medical device does not contain a medicinal substance, including derivatives of human blood or plasma, tissues or cells of human origin or derivatives thereof, tissues or cells of animal origin or derivatives thereof.

Disposal:

Compression stockings can be disposed of with normal household waste or recycled as a textile. Although the product is intended for use on healthy, intact skin, it is possible that contamination with body fluids may have occurred during use. In this case, dispose of the compression stockings as infectious material.

The material used is non-toxic as long as it does not burn. Never burn used compression stockings.

Any serious adverse event occurring in connection with this medical device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

List of abbreviations:

KT, CCL – compression class
 mmHg – millimeters of mercury column (unit of blood pressure measurement on the leg)

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