41092

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(EN) INSTRUCTIONS FOR USE - Diafit medical socks

Dear customer,

The Diafit medical socks you hold in your hands have been specially developed for patients with diabetes and those with sensitive skin on their legs. You can maximize the benefits of this product by its proper care and use. Diafit medical socks are classified as a medical device.

Diafit CLASSIC medical socks with their high percentage of natural long-staple combed cotton are comfortable and at the same time sturdy and durable. The socks have a flat, chain-stitched toe, thus reducing seam blisters in the toe area. A lightweight subtalar gusset knitted at the instep keeps the sock from bunching. The gentle-fit hem is comfortable and does not pinch, yet has enough grip to prevent the sock from slipping down.

Diafit PREMIUM medical socks contain a high percentage of natural long-staple combed cotton. That makes them comfortable, yet strong and durable, thanks to the long fibres that overlap sufficiently in the knit. The socks have a flat, chain-stitched toe, thus reducing seam blisters in the toe area. The footbed is reinforced with a plush loop for greater thermal comfort and an even softer tread. A lightweight subtalar gusset knitted at the instep keeps the sock from bunching. The gentle-fit hem is comfortable and does not pinch, yet has enough grip to prevent the sock from slipping down.

Diafit CLASSIC and Diafit PREMIUM are seamless medical socks with a gentle-fit hem. They were made by circular knitting technology. The materials used in their production are non-irritating and latex-free. The therapeutic effect of medical socks is to protect the foot, maintain thermal comfort in the foot area, and prevent blisters. For the desired effect, the correct size must be chosen. Diafit CLASSIC and Diafit PREMIUM are medical devices intended for use both at home and in medical care facilities.

Name of the medical device:

Diafit CLASSIC medical socks

Medical device variants:

- size 36-39, 39-42, 41-44, 44-47

Diafit PREMIUM medical socks

Medical device variants:

- size 36-39, 39-42, 41-44, 44-47

User profile:

Diafit CLASSIC medical socks and Diafit PREMIUM medical socks are medical devices intended for use at home and in health care facilities.

Intended use of the medical device:

A medical device designed for diabetics and patients with sensitive skin on their lower limbs.

Indications:

- Diabetes mellitus (diabetic foot syndrome)
- Lower limb sensitivity disorders (e.g. peripheral neuropathy)
- Sensitive skin in the foot area of the lower limbs

If using medical socks for the first time, consult with your doctor.

Contraindications:

- Acute weeping skin manifestations
- Irritated or otherwise broken skin
- Hypersensitivity to materials used

Your doctor should consider the use of any medical device in view of the known contraindications and possible risks compared to the therapeutic benefits.

Patient target group:

Patients with the above indications. Medical socks are intended for use on healthy, intact skin. This medical device is suitable for all age groups and both sexes. There are no restrictions for use during pregnancy.

Side effects:

Side effects may occur especially if the size is not chosen correctly, resulting in blisters where the sock material bunches up. Sensitive and hypersensitive individuals may experience skin irritation (redness, itching, or blistering) in places where the device material comes into contact with the skin.

Fitting your medical devices:

For the desired effect, the right size must be used. Size is determined by foot size. Diafit CLASSIC and Diafit PREMIUM medical socks are available in four sizes.

Correctly chosen size and use:

- Medical socks must be worn evenly, without folds and bunching up at any point.
- Increased care of the feet (rough skin on the heels) will reduce the risk of damaging the medical device and will prolong its lifespan.
- In the event of visible mechanical damage to the product, stop wearing it and replace it with a new one.

Care and maintenance:

Diafit CLASSIC and Diafit PREMIUM medical devices are medical devices intended for repeated use by a single user. This product may be used safely and effectively for 6 months from the first use if the following guidelines are respected:

- Medical socks can be washed in a washing machine (maximum wash temperature 30°C, gentle cycle), preferably in a protective cover (wash bag)
- Do not use fabric softener
- Lay flat to dry away from sources of direct heat (radiator, sunlight)
- Do not iron medical devices
- Maximum number of washing cycles is 120
- Store in a dark, dry place in the original packaging if possible
- Do not make mechanical changes to the medical device, do not repair areas damaged by use, do not trim the edges
- Keep the medical device away from organic solvents and bleaching agents

This medical device may be used safely for 60 months from the date of manufacture. The expiry date is marked on the packaging of the medical device.

Care symbols:

Do not use fabric softener

When not to use:

Discontinue use of the medical device immediately if:

- There are signs of mechanical damage – replace with new device
- You notice a deterioration in your health that may be associated with the use of the medical device – consult your doctor in such case

Composition:

Diafit CLASSIC medical socks 80% cotton, 18% PA, 2% elastane

Diafit PREMIUM medical socks 83% cotton, 15% PA, 2% elastane

This medical device does not contain a medicinal substance, including derivatives of human blood or plasma, tissues or cells of human origin or derivatives thereof, tissues or cells of animal origin or derivatives thereof.

Disposal:

Medical devices can be disposed of with normal household waste or in a textile container. Although the product is intended for use on healthy, intact skin, it is possible that contamination with body fluids may have occurred during use. In such a case, dispose of the medical device as infectious material.

The material used is non-toxic as long as it does not burn. Never incinerate used medical devices.

Any serious adverse event occurring in connection with this medical device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

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